

Product overview

www.sorbion.com



In conjunction with targeted medical treatment, sorbion wound dressings help to relieve symptoms and improve the quality of life of patients with acute and chronic wounds.




sorbion GmbH & Co. KG

is a modern, innovative healthcare company, specialising in the development, production and sales of products for modern wound care. We aim to improve the quality of life of patients with acute and chronic wounds while, at the same time, meeting the clinical and economic requirements of modern wound care. sorbion offers solutions for today's patient care and that of tomorrow – locally and tailored to patient needs. sorbion products promote effective patient care in the various phases of wound healing. Our long-term interprofessional and user-centred cooperation with everyone involved in the provision of healthcare ensures a shared commitment to excellent health service, cost-effectiveness and evidence-based patient care.

Contact us – we'll be delighted to hear from you!

Content

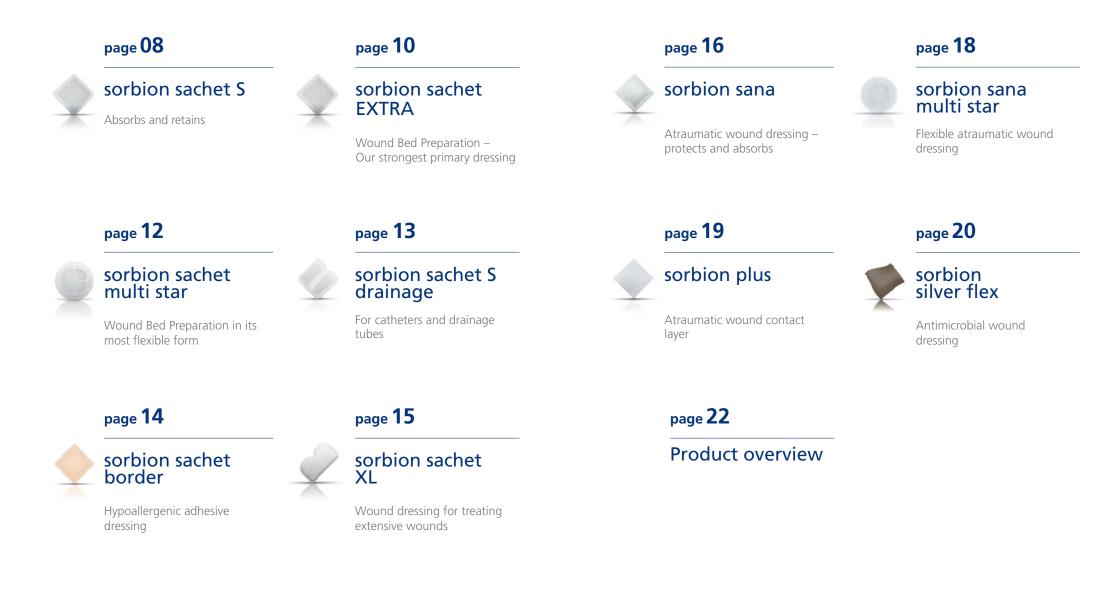

1

| Heavily ex | cuding | Highly to exuding | moderately | Moderate exuding | ly to low |
|-------------|--------|-------------------|------------|---------------------|-----------|
| Superficial | Deep | Superficial | Deep | Superficial | Deep |
| wounds | wounds | wounds | wounds | wounds | wounds |

T

Cleaning phase

Infection

| Primary dressing | sorbion silver flex | sorbion silver flex | sorbion silver flex | sorbion silver flex | sorbion silver flex | sorbion silver flex |
|--|--|----------------------------------|--|----------------------------------|------------------------|--------------------------------|
| Secondary dressing | sorbion sachet EXTRA; sorbion sachet XL | sorbion sachet multi star | sorbion sachet S; sorbion sachet XL; sorbion sachet border | sachet multi star | | |
| Necrosis | | | Soraci | | | |
| * In conjunction with a suitable primary dressing | sorbion sachet EXTRA* | sorbion sachet multi star* | sorbion sachet S*; sorbion sachet border* | sorbion sachet multi star* | sorbion sana* | sorbion sana multi star* |
| Fibrin | | | | | | |
| Primary dressing | sorbion sachet EXTRA | sorbion sachet multi star | sorbion sachet S; sorbion sachet border | sorbion sachet multi star | sorbion sana | sorbion sana multi star |

1

Granulation phase

Granulation

| Primary dressing | - | — | sorbion sana | sorbion sana multi star | sorbion plus | sorbion plus |
|--------------------|---|---|---|----------------------------|--------------|--------------------------|
| Secondary dressing | _ | — | sorbion sachet border, if combined with sorbion plus | sachet border | | sorbion sachet border |

Epithelial phase

| Formation of intact epithelial tissue | _ | — | _ | — | sorbion plus | — |
|---|---|---|---|---|--------------|---|

HRT

Performance with no compromises



Hydration Response Technology (HRT) is the design concept we use to develop and manufacture state-of-the-art wound care products at sorbion.


Our aim is to make products that deliver maximum performance, using as few components as possible and only hypoallergenic materials. One of the advantages of using modern mechanical manufacturing techniques is that we can dispense with the use of glues, binders and adhesives. This enables us to develop highly biocompatible wound dressings for demanding applications, e. g. for patients who are susceptible to skin sensitisation.

Hydrokinetic fibres are a key element in implementing this design concept in our products. These fibres form a matrix characterised by a specific combination of materials, including their proportions, density and mass. The production process involves binding together selected cellulose fibres (A) and gel-forming polymers (B) following a special formula.

The expansion border, which enables wide contact with the wound region, even at high saturation rates, also plays an important role in the design concept.

The soft, flexible polypropylene envelope is ultrasonically sealed and can therefore be manufactured without the use of binders or adhesives. It has a high liquid penetration rate and can be removed without leaving any residue. It also has a high tear strength, even when saturated, and a high moisture vapour transmission rate (MVTR).

sorbion sachet

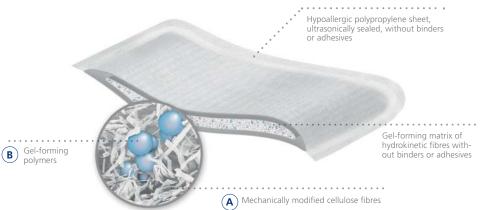

sorbion sachet S Absorbs and retains

sorbion sachet S is a versatile fibre dressing with high wearing comfort for primary wound contact, which

- absorbs wound exudate and locks it in removes wound debris
- absorbs and sequesters wound bacteria and reduces bioburden
- maintains a moist wound environment.

Indications

The dressing is suitable for moderately to highly exuding wounds, including

- acute and chronic wounds
- leg, foot and pressure ulcers
- wounds healing by secondary intention.

Key benefits

Proactive exudate management

- Creates a moist wound environment by gel formation
- Large capacity for wound exudate
- High retention of absorbed fluids
- Reduces oedema

Wound bed preparation

- Reduction of fibrinous necrotic tissue (soft debridement)
- Modulation of proteases and proinflammatory cytokines
- Management of wound debris



- Powerful antimicrobial properties inside the dressing especially against Pseudomonas
- Effective against a large number of pathogenic germs
- Reduces the risk of cross-contamination by locking in bacteria

High wearing comfort

- Securely locks in exudate, even under compression therapy
- Reduces odour
- Low risk of skin irritation and sensitization

| Item name | Items per unit | ltem no. |
|---------------------------|----------------|-------------|
| sorbion sachet S | 10 | 22143002-10 |
| 7,5 x 7,5 cm 3 x 3 inch | 25 | 22143002-25 |
| | 50 | 22143002-50 |
| sorbion sachet S | 10 | 22143003-10 |
| 12 x 5 cm 5 x 2 inch | 25 | 22143003-25 |
| | 50 | 22143003-50 |
| sorbion sachet S | 10 | 22143004-10 |
| 10 x 10 cm 4 x 4 inch | 25 | 22143004-25 |
| | 50 | 22143004-50 |
| sorbion sachet S | 10 | 22143009-10 |
| 20x10 cm 8x4 inch | 25 | 22143009-25 |
| | 50 | 22143009-50 |
| sorbion sachet S | 10 | 22143016-10 |
| 15 x 15 cm 6 x 6 inch | 25 | 22143016-25 |
| | 50 | 22143016-50 |
| sorbion sachet S | 10 | 22143006-10 |
| 20x20 cm 8x8 inch | 25 | 22143006-25 |
| | 50 | 22143006-50 |
| sorbion sachet S | 10 | 22143007-10 |
| 30 x 20 cm 12 x 8 inch | 25 | 22143007-25 |
| | 50 | 22143007-50 |



sorbion sachet EXTRA

Our strongest primary dressing

sorbion sachet EXTRA is a versatile fibre dressing with high wearing comfort for primary wound contact, which

- absorbs wound exudate and locks it in
- removes wound debris
- absorbs and sequesters wound bacteria and reduces bioburden.
- maintains a moist wound environment

sorbion sachet EXTRA provides even greater absorbency, retention and ability to bind harmful wound constituents than sorbion sachet S. With its outstanding capacity and resulting performance, sorbion sachet EXTRA is ideal for wound situations where wound bed conditioning and a particular wear time are required (up to 4 days).

Indications

sorbion sachet EXTRA is used for moderately to highly exuding wounds, e.g.

- Venous leg ulcers/mixed aetiology ulcers
- Decubitus
- Diabetic foot syndrome
- Secreting wounds
- Discharging laparotomy wounds healing by secondary intention
- Secreting burn areas
- Purulent fistulas to the skin

- Drainage
- Acute wounds
- Post-interventional wounds
- Split-thickness skin graft donor sites
- Abdominal dehiscence
- Ulcerated carcinomas and similar secreting wounds

Key benefits

Proactive exudate management

- Creates a moist wound environment by gel formation
- Large capacity for wound exudate
- High retention of absorbed fluids
- Reduces oedema

Wound bed preparation

- Reduction of fibrinous necrotic tissue (soft debridement)
- Modulation of proteases and proinflammatory cytokines
- Management of wound debris

Reduction of bacterial bioburden

- Powerful antimicrobial properties inside the dressing especially against Pseudomonas
- Effective against a large number of pathogenic germs
- Reduces the risk of cross-contamination by locking in bacteria

High wearing comfort

- Securely locks in exudate, even under compression therapy
- Reduces odour
- Low risk of skin irritation and sensitization

Treatment with sorbion sachet has proven to be an efficient and cost-effective alternative to negative-pressure wound therapy for a variety of wound situations^{1,2,3,4}.

1 Hermans, M. & Kwon Lee, S. (2012): Hydration Response Technology for the management of NPWT-indicated wounds: results of a series of 6 cases. Poster presented at SAWC, Spring

2012. 2 2 Cutting, K., Hermans, M., Kwon Lee, S., Wolcott, R. & Treadwell, T. (2011): Key considerations in choice of wound management therapeutics between an advanced wound care dressing utiliaing Hydration Response Technology or a durable medical device NPNT. Health Directions, Chorleywood, Hertfordshire, UK. 3 Riepe, G. & & Konsteing K. (2011): Uniter Druck? Line hydrabitive Alternative zur Vakumtherapie In: Wundhanagement, H. Sonderheft 2/2011, S. 12–17. 4 Ling, L. (2011). Sorbion Dressing: Using One dressing Product to Manage a Variety of Symptoms Affecting Wounds. Poster presented at EWMA 2011, Brussels.

| Item name | ltems per unit | ltem no. |
|--|----------------|-------------|
| sorbion sachet EXTRA 5x5 cm 2x2 inch | 5 | 22303011-05 |
| sorbion sachet EXTRA 7,5 x 7,5 cm 3 x 3 inch | 5 | 22303002-05 |
| sorbion sachet EXTRA 10 x 10 cm 4 x 4 inch | 5 | 22303004-05 |
| sorbion sachet EXTRA 20 x 10 cm 8 x 4 inch | 5 | 22303009-05 |
| sorbion sachet EXTRA 20 x20 cm 8x8 inch | 5 | 22303006-05 |
| sorbion sachet EXTRA 30 x 20 cm 12 x 8 inch | 5 | 22303007-05 |



sorbion sachet multi star

Wound Bed Preparation in its most flexible form

sorbion sachet multi star combines the advantages of sorbion sachet with extended applications. With its unique shape sorbion sachet multi star allows flexible applications for use – on sites such as the toes, heel, foot, sacrum, chest, axillae, elbows or wound cavities.



sorbion sachet

sotion

nultistar

| Item name | ltems per unit | Item no. |
|---------------------------|----------------|-------------|
| sorbion sachet multi star | 5 | 22143019-05 |
| ø 8 cm 3.5 inch | 10 | 22143019-10 |
| sorbion sachet multi star | 5 | 22143020-05 |
| ø 14 cm 5.5 inch | 10 | 22143020-10 |

sorbion sachet S drainage For catheters and

drainage tubes

sorbion sachet S drainage was developed for use with catheters, drainages, etc. in particular. The slit design ensures an especially good fit at the exit points of tubes, etc.









| Item name | Items per unit | ltem no. |
|---------------------------|----------------|-------------|
| sorbion sachet S drainage | 10 | 22143008-10 |
| | 25 | 22143008-25 |
| | 50 | 22143008-50 |
| | | |

sorbion sachet border


Hypoallergenic adhesive dressing

sorbion sachet border is a self-adhesive hydroactive wound dressing for the management of moderately to highly exuding wounds. Used wherever quick and easy application is required, without sacrificing the well-proven benefits of sorbion sachet S.



Key benefits

- Quick and easy application
- Hypoallergenic

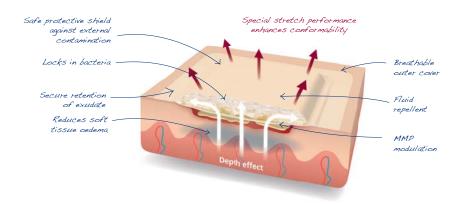

Flexible and breathable

Excellent absorption and retention

| Item name | ltems per unit | Size | Pad Size | ltem no. |
|--|----------------|--------------------------|--------------------------|----------------------------|
| sorbion sachet border | 10 | 10 x 10 cm | 7,5x7,5 cm | 22663011-10 |
| 10x10 cm 4x4 inch | 25 | 10 x 10 cm | 7,5x7,5 cm | 22663011-25 |
| sorbion sachet border | 10 | 15 x 15 cm | 10 x 10 cm | 22663004-10 |
| 15x15 cm 6x6 inch | 25 | 15 x 15 cm | 10 x 10 cm | 22663004-25 |
| sorbion sachet border 25x15 cm 10x6 inch | 10 25 | 25 x 15 cm 25 x 15 cm | 20 x 10 cm 20 x 10 cm | 22663009-10 22663009-25 |

sorbion sachet XL

Wound dressing for treating extensive wounds

The shape and size of **sorbion sachet XL** has been specially designed to treat moderately to highly exuding difficult-to-dress wounds on anatomical sites such as: Lower leg, sacral region, thorax, thigh, lower part of the abdomen.



Key benefits

• More convenient with the use of a single dressing

Protection against maceration and skin irritation

Long lasting patient comfortEasy to apply



| Item name | Items per unit | Item no. | |
|---|----------------|-------------|--|
| sorbion sachet XL 45x25 cm 18x10 inch | 10 | 22143017-10 | |

sorbion sana

Atraumatic wound dressing – protects and absorbs

sorbion sana is an atraumatic fibre dressing for primary wound contact with high wearing comfort that

- maintains a moist wound environment
- promotes tissue formation
- absorbs and safely locks away exudate
- permits pain-free dressing changes.

Indications

sorbion sana is suitable for low to moderately exuding wounds, including:

- Acute or chronic wounds (e.g. wounds healing by secondary intention or iatrogenic wounds)
- Leg ulcers
- Decubitus ulcers
- Diabetic foot syndrome

sorbion sana

8.5×8.5 cm - 15×3.5 inch

sorbion

Trauma wounds

Burn wounds

Fungating wounds



Safely binds exudate

- Gel formation creates a moist wound environment
- Extremely large capacity for wound exudate
- High retention of absorbed fluids

Patient/User protection


- Protection against adhesion
- Functional clothing protection
- Can be removed without leaving any residue
- Hypoallergenic

High wearing comfort

- Odour control
- Low risk of skin irritation and allergies
- Securely locks in exudate, even under compression therapy

...and, in addition, sorbion sana is free from binders, glues, fats/paraffins and latex.



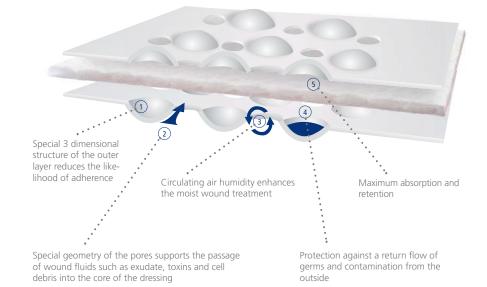

| Item name | ltems per unit | Item no. |
|-------------------------------|----------------|-------------|
| sorbion sana | 10 | 25523002-10 |
| 8,5 x 8,5 cm 3.5 x 3.5 inch | 25 | 25523002-25 |
| | 50 | 25523002-50 |
| sorbion sana | 10 | 25523004-10 |
| 12 x 12 cm 5 x 5 inch | 25 | 25523004-25 |
| | 50 | 25523004-50 |
| sorbion sana | 10 | 25523009-10 |
| 22x12 cm 9x5 inch | 25 | 25523009-25 |
| | 50 | 25523009-50 |
| sorbion sana | 10 | 25523006-10 |
| 22 x 22 cm 9 x 9 inch | 25 | 25523006-25 |
| | 50 | 25523006-50 |

sorbion sana multi star

Flexible atraumatic wound dressing

sorbion sana multi star combines the advantages of sorbion sana with extended applications. With its unique shape sorbion sana multi star allows flexible applications for use on sites – such as the toes, heel, foot, sacrum, chest, axillae, elbows, or wound cavities.



sorbion sana

| Item name | Items per unit | ltem no. |
|-------------------------|----------------|-------------|
| sorbion sana multi star | 5 | 25143019-05 |
| ø 11 cm 4.5 inch | 10 | 25143019-10 |
| sorbion sana multi star | 5 | 25143020-05 |
| ø 17 cm 7 inch | 10 | 25143020-10 |

sorbion plus

Atraumatic wound contact layer

The primary atraumatic wound contact layer **sorbion plus** consists of a hypoallergenic polyethylene film. Its special three-dimensional structure minimises contact with the wound bed. The pores of the wound contact layer conduct fluids from the wound bed to the secondary dressings. The three-dimensional structure also reduces the flow of fluids back into the wound. **sorbion plus** moulds perfectly to the wound surface and can be removed without leaving any residue.

Indications

Use as wound contact layer in combination with an appropriate wound dressing for exuding wounds such as chronic wounds (e.g. leg ulcers, diabetic foot syndrome, decubitus ulcers), exuding burn areas, post-operative wounds and dehiscence, fistulas and ulcerating carcinomas. Also as a wound contact layer for treating acute wounds such as skin abrasions, superficial burns and surgical wounds.

Key benefits

- Prevents adhesion to the wound
- Atraumatic dressing change
- Supports the drainage of wound fluids
- Free from fats and allergens

| tem name | Items per unit | Item no. |
|-------------------------|----------------|-------------|
| orbion plus | 10 | 33143002-10 |
| 5x7,5 cm 3x3 inch | 25 | 33143002-25 |
| | 50 | 33143002-50 |
| orbion plus | 10 | 33143004-10 |
| 0x10 cm 4x4 inch | 25 | 33143004-25 |
| | 50 | 33143004-50 |
| orbion plus | 10 | 33143009-10 |
| 20x10 cm 8x4 inch | 25 | 33143009-25 |
| | 50 | 33143009-50 |
| orbion plus | 10 | 33143006-10 |
| 0x20 cm 8x8 inch | 25 | 33143006-25 |
| | 50 | 33143006-50 |
| orbion plus | 10 | 33143007-10 |
| 0 x 20 cm 12 x 8 inch | 25 | 33143007-25 |
| | 50 | 33143007-50 |



sorbion silver flex

Antimicrobial wound dressing

sorbion silver flex powered by Silverlon[®] is an antimicrobial wound dressing for advanced wound care, as well as in combination with negative-pressure wound therapy for the following indications:

- Diabetic, pressure and venous ulcers,
- Skin graft donor and mesh graft sites
- Traumatic and surgical wounds
- Infected wounds*



*sorbion silver flex may be used on those infected wounds listed above

Product design

- Large active surface area due to many individual nylon fibres
- Each individual fibre is permanently coated with 99% pure silver
- sorbion silver flex continuously releases large quantities of silver ions

Additionally...

- Optimal adaption to the wound bed
- Supports the drainage of wound fluids into the secondary dressing
- Atraumatic dressing change

Mode of action

- Quick and continuous release of silver ions on contact with moisture
- Bactericidal and fungicidal effect demonstrated on over 360 bacteria and fungi such as MRSA, VRE and C. albicans¹
- Prevents and reduces unpleasant odour

Key benefits

A safe way of treatment

- No deposits of heavy metal, elemental silver into the wound
- Unrestricted wound assessment as the wound is not discoloured
- Atraumatic dressing change if used as indicated
- No known resistances in over 10 years of application experience

1 Heggers, J. P. et al, Journal of Burns, Acticoat[®] vs. Silverlon[®]. The Truth, Vol. 1, 2002 Z Karr et al, Wound Healing and Cost-Saving Benefits of Combining Negative-Pressure Wound Therapy with Silver, advances in skin & wound care Vol. 26 No. 12, 2013 3 Marcet et al, 2011, The use of Silver Nylon in Preventing Surgical Site Infections Following Colon and Rectal Surgery, Diseases of the Colon & Rectum, Vol. 54, No. 8 4 Carson et al, Silver Dressings Used With Wound Vacuum Assisted Closure: Is There an Advantage?, 2005, presented at SAWC



Highly effective

- Bactericidal and fungicidal effect demonstrated on over 360 bacteria and fungi such as MRSA, VRE and C. albicans¹
- Rapid onset of action for combating pathogens
- Long-lasting effect due to continuous release of large and effective amounts of silver ions

Economic treatment

- Up to 7 days wear time
- Reduces frequency of dressing changes and effort incurred in treatment^{2,3,4}
- Can be used in combination with all secondary dressings currently in use as well as with negative pressure wound therapy
- Only the secondary dressing needs to be changed, while sorbion silver flex can remain in place

| Item name | | ltems per unit | ltem no. |
|---|-----|----------------|-----------|
| sorbion silver flex 5x5 cm 2x2 inch | NEW | 5 | WCD22-NPS |
| sorbion silver flex 10x10 cm 4x4 inch | NEW | 5 | WCD44-NPS |
| sorbion silver flex 20x10 cm 8x4 inch | NEW | 5 | WCD48-NPS |
| sorbion silver flex 20x20 cm 8x8 inch | NEW | 5 | WCD88-NPS |

sorbion sachet S

| Item name | Items per unit | ltem no. |
|--|----------------|-------------|
| sorbion sachet S | 10 | 22143002-10 |
| 7,5 x 7,5 cm 3 x 3 inch | 25 | 22143002-25 |
| | 50 | 22143002-50 |
| sorbion sachet S | 10 | 22143003-10 |
| 12x5 cm 5x2 inch | 25 | 22143003-25 |
| | 50 | 22143003-50 |
| sorbion sachet S | 10 | 22143004-10 |
| 10x10 cm 4x4 inch | 25 | 22143004-25 |
| | 50 | 22143004-50 |
| sorbion sachet S | 10 | 22143009-10 |
| 20x10 cm 8x4 inch | 25 | 22143009-25 |
| | 50 | 22143009-50 |
| sorbion sachet S | 10 | 22143016-10 |
| 15x15 cm 6x6 inch | 25 | 22143016-25 |
| | 50 | 22143016-50 |
| sorbion sachet S 20x20 cm 8x8 inch | 10 | 22143006-10 |
| 20x20 cm 8x8 inch | 25 | 22143006-25 |
| | 50 | 22143006-50 |
| sorbion sachet S | 10 | 22143007-10 |
| 30x20 cm 12x 8 inch | 25 | 22143007-25 |
| | 50 | 22143007-50 |
| sorbion sachet S | 10 | 22143008-10 |
| drainage 8x8 inch | 25 | 22143008-25 |
| | 50 | 22143008-50 |
| sorbion sachet XL 45x25 cm 18x10 inch | 10 | 22143017-10 |

sorbion sachet EXTRA

| Item name | ltems per unit | ltem no. |
|--|----------------|-------------|
| sorbion sachet EXTRA 5x5 cm 2x2 inch | 5 | 22303011-05 |
| sorbion sachet EXTRA 7,5 x7,5 cm 3 x 3 inch | 5 | 22303002-05 |
| sorbion sachet EXTRA 10x10 cm 4x4 inch | 5 | 22303004-05 |
| sorbion sachet EXTRA 20x10 cm 8x4 inch | 5 | 22303009-05 |
| sorbion sachet EXTRA 20x20 cm 8x8 inch | 5 | 22303006-05 |
| sorbion sachet EXTRA 30x20 cm 12x8 inch | 5 | 22303007-05 |

sorbion sachet multi star

| Item name | Items per unit | ltem no. |
|---|----------------|-------------|
| sorbion sachet multi star ø 8 cm 3.5 inch | 5 | 22143019-05 |
| | 10 | 22143019-10 |
| sorbion sachet multi star ø 14 cm 5.5 inch | 5 | 22143020-05 |
| | 10 | 22143020-10 |

sorbion sachet border

| Item name | Items per unit | Size | Pad Size | ltem no. |
|---|----------------|------------|------------|-------------|
| sorbion sachet border | 10 | 10 x 10 cm | 7,5x7,5 cm | 22663011-10 |
| 10x10 cm 4x4 inch | 25 | 10 x 10 cm | 7,5x7,5 cm | 22663011-25 |
| sorbion sachet border 15x15 cm 6x6 inch | 10 | 15 x 15 cm | 10 x 10 cm | 22663004-10 |
| | 25 | 15 x 15 cm | 10 x 10 cm | 22663004-25 |
| sorbion sachet border 25x15 cm 10x6 inch | 10 | 25 x 15 cm | 20 x 10 cm | 22663009-10 |
| | 25 | 25 x 15 cm | 20 x 10 cm | 22663009-25 |

sorbion sachet S











| sorbion sana | | | |
|--|----------------|-------------|--|
| Item name | Items per unit | ltem no. | |
| sorbion sana 8,5x8,5 cm 3.5x 3.5 inch | 10 | 25523002-10 | |
| 6,5x6,5 cm 5.5x 5.5 inch | 25 | 25523002-25 | |
| | 50 | 25523002-50 | |
| sorbion sana | 10 | 25523004-10 | |
| 12x12 cm 5x5 inch | 25 | 25523004-25 | |
| | 50 | 25523004-50 | |
| sorbion sana 22x12 cm 9x5 inch | 10 | 25523009-10 | |
| 22212 011 923 1101 | 25 | 25523009-25 | |
| | 50 | 25523009-50 | |
| sorbion sana 22x22 cm 9x9 inch | 10 | 25523006-10 | |
| 22222 011 929 1101 | 25 | 25523006-25 | |
| | 50 | 25523006-50 | |



sorbion sana multi star

| Item name | ltems per unit | ltem no. |
|---|----------------|-------------|
| sorbion sana multi star | 5 | 25143019-05 |
| ø 11 cm 4.5 inch | 10 | 25143019-10 |
| sorbion sana multi star ø 17 cm 7 inch | 5 | 25143020-05 |
| | 10 | 25143020-10 |

sorbion plus

| Item name | Items per unit | ltem no. |
|---------------------------|----------------|-------------|
| sorbion plus | 10 | 33143002-10 |
| 7,5 x 7,5 cm 3 x 3 inch | 25 | 33143002-25 |
| | 50 | 33143002-50 |
| sorbion plus | 10 | 33143004-10 |
| 10x10 cm 4x4 inch | 25 | 33143004-25 |
| | 50 | 33143004-50 |
| sorbion plus | 10 | 33143009-10 |
| 20x10 cm 8x4 inch | 25 | 33143009-25 |
| | 50 | 33143009-50 |
| sorbion plus | 10 | 33143006-10 |
| 20x20 cm 8x8 inch | 25 | 33143006-25 |
| | 50 | 33143006-50 |
| sorbion plus | 10 | 33143007-10 |
| 30x20 cm 12x8 inch | 25 | 33143007-25 |
| | 50 | 33143007-50 |
| | | |

sorbion silver flex

| Item name | | Items per unit | ltem no. |
|--|-----|----------------|-----------|
| sorbion silver flex | NEW | 5 | WCD22-NPS |
| 5x5 cm 2x2 inch | | - | |
| 10x10 cm 4x4 inch | NEW | 5 | WCD44-NPS |
| sorbion silver flex 20x10 cm 8x4 inch | NEW | 5 | WCD48-NPS |
| sorbion silver flex 20x20 cm 8x8 inch | NEW | 5 | WCD88-NPS |

Product availability may differ from country to country.









sorbion GmbH & Co.KG Im Südfeld 11 | 48308 Senden | Germany Tel. +49 2536 34400-100 | Fax +49 2536 34400-110 info@sorbion.com | www.sorbion.com